

INSTRUCTION FOR USE OF PERSONAL PROTECTIVE EQUIPMENT

MERCATOR simple vinyl

The instruction below should be used in conjunction with detailed information on the packaging.

Description of the product


Protective gloves (for minimal risk only), vinyl, powder-free or powdered, for single use, non-sterile

Colours	: black / blue / clear
Sizes	: XS (5-6), S (6-7), M (7-8), L (8-9), XL (9-10)
Quantity in packaging	: 100 pcs. by weight
Shelf life	: 3 or 5 years depends on LOT number (check the packaging)

Storage instructions

Do not expose to direct sunlight, ozone sources or sources of fire. Store in a dry and cool place, at a temperature of 5-35°C. Do not keep in direct vicinity of solvents, oils, fuels and lubricants.

Food contact

Gloves are marked with food contact symbol  and comply with the requirements of Regulation (EU) No 10/2011, European Regulation (EC) No 1935/2004 and with Regulation (EC) No 2023/2006 on Good Manufacturing Practice. Gloves are suitable for handling the food and have been tested for Overall Migration Test acc. EN 1186.

Intended use

These are protective gloves, vinyl, for single use, non-sterile. Gloves are classified as a Personal Protective Equipment Category I, for minimal risk only. Gloves designed to protect against minimal risks, i.e. prolonged contact with water. Their design and labelling corresponds to the requirements of the European Regulation 2016/425 on Personal Protective Equipment. Gloves should be used solely according to their intended application.

Manufacturer

MERCATOR MEDICAL S.A.
ul. H. Modrzejewskiej 30
31-327 Kraków, Polska

Declaration of Conformity and this instruction for use available under below web address:
<https://mercatormedical.eu>

Precautions and indications for use

Dry hands before taking the gloves out from the packaging. Before usage, inspect the gloves for any defect or imperfections. If the gloves get punctured, torn or broken during their use, take them off and put on the new ones. Avoid using gloves dirty in the inside as they may cause irritation leading to skin inflammation or more serious damages.

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on the temperature, abrasion and degradation. The gloves should not be used in contact with open fire and to protect against any sharp tools. The gloves are not intended for welding, electric shock protection, ionizing radiation or from the effect of hot or cold objects.






















Components / hazardous components

Components used in making gloves may cause allergic reactions in some people. Some gloves may contain components known to be a possible cause of allergy for person allergic to them, who may develop contact irritation and/or allergic reaction. In case of an allergic reaction consult a doctor.

Disposal

Dispose of in accordance with applicable regulations.

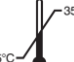Symbols used on the packaging

	Manufacturer		Expiry date		Product quality is not ensured if the package is damaged		VINYL	Vinyl gloves
	Lot / batch number		Date of manufacture		Recyclable packaging			Powder-free gloves (depends on the type)
	Catalogue number		Keep away from ozone		Package can be treated as municipal waste			Powdered gloves (depends on the type)
	Indicates compliance with the requirements of Ukrainian market		Keep dry		Suitable for food contact			For single use only
	Model number		Keep away from sunlight		Temperature limitation		NON STERILE	Non-sterile
								Consult instructions for use

MERCATOR simple vinyl MODEL #0010

REF NO: RP21015001-05_0010; RP20016001-05_0010; RP20017001-05_0010

PPE classification & compliance

Gloves are category I, for minimal risk only Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

EN ISO 21420:2020, EN ISO 374-2:2019.



MERCATOR simple vinyl MODEL #1537

REF NO: RP21015001-05_1537; RP20016001-05_1537; RP20017001-05_1537; RP20018001-05_1537

PPE classification & compliance

Gloves are category I, for minimal risk only Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

EN ISO 21420:2020, EN ISO 374-2:2019.



MERCATOR simple vinyl MODEL #1056

REF NO: RP20016001-05_1056; RP20017001-05_1056


PPE classification & compliance

Gloves are category I, for minimal risk only Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

EN ISO 21420:2020, EN ISO 374-2:2019.



How to put the gloves on?

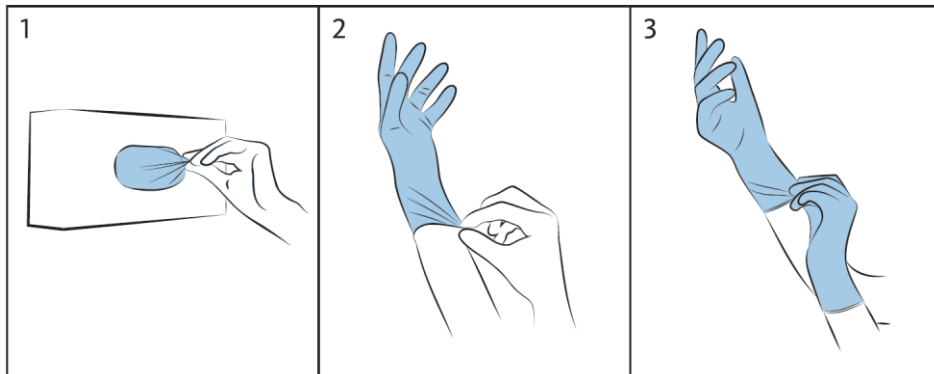

How to take the gloves off?

